'PRODUCT NAME EYETEC® DIAMOND KNIFE CLEANING BLOCK
'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE

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This Instructions for use relates to the following products:

- GB - This Instructions for use relates to the following products: **Diamond Knife Cleaning Block**
- DE - Diese Gebrauchsanweisung bezieht sich auf die folgenden Produkte: **Diamantmesser-Abstreifblock**
- ES - This instructions for use relates to the following products : **Almohadilla limpiadora para cuchilla de diamante**
- FR - Mode d’emploi des produits suivants : **Bloc de nettoyage couteaux diamants**
- IT - Le seguenti istruzioni per l’uso si riferiscono ai seguenti prodotti : **Blocko pulisci bisturi diamantato**
- PT - Estas instruções referem-se aos seguintes produtos : **Bloco de limpeza de lâminas de diamante**

GB - INSTRUCTIONS FOR USE

Product Description:

Diamond Knife Cleaning Blocks are manufactured from 100% biocompatible PVA sponge, and are designed to allow the surgeon to perform two specific operations.

- The removal of debris from the diamond blade following and during (if required) the surgical procedure.
- The polishing of the blade

The block softens when moistened with sterile water for cleaning. The dry material is used for polishing the diamond.

Intended Population / Patient Target Groups:

Diamond knife cleaning blocks are not intended to be used on patients.

Intended Users:

Healthcare professionals only; trained ophthalmic surgeons and ophthalmic theatre staff including trained decontamination professionals.

PRODUCT NAME EYETEC® DIAMOND KNIFE CLEANING BLOCK
PRODUCT INFORMATION AND INSTRUCTIONS FOR USE

Intended Purpose:

- Statement for Intended purpose for diamond knife cleaning block

A non-contact, single-use block of PVA intended to be used in a sterile condition by healthcare professionals to help clean and polish a medical diamond knife blade prior to full decontamination. The diamond knife cleaning block is only to be handled with medical grade gloves and as such is non-contact with a cumulative transient use of less than 60 minutes. There are no known contraindications for this device. These devices are not intended to be used on a patient.

Indications for use:

Diamond knives are expensive and very easy to damage. The diamond knife cleaning block has been specially developed for the maintenance and care of these delicate surgical products.

Contraindications:

Contraindicated for patient use.

Side effects:

No known side effects

Clinical benefits, see also Intended Purpose:

Diamond knife cleaning blocks are not used in a clinical application. Devices are supplied sterile, single use only, declared 5-year shelf life.

CAUTIONS:

- These are delicate medical devices and should be handled with care. Do not apply excessive forces.
- Medical grade (e.g. CE marked) surgical gloves should be used to handle these devices, cross-contamination and infection risks due to unwanted exposure of bacteria, prion, fungus or virus.
- PVA products are to be stored out of direct sunlight and at normal room temperature, or products may expand compromising the pouch seal.
- Do NOT re-sterilise or re-use. Devices are supplied sterile by irradiation ready for use.
- Sterility and performance are not guaranteed on devices that have passed their Use By date or if the packaging or seals have been damaged. If label including use by date is illegible, do not use.
- This device is intended for use by trained medical persons possessing the requisite skill and experience to use the device in accordance with the prevailing standards of medical practice and in conjunction with the instructions for this device.
- As with all PVA products loose particles may occur, these should be rinsed from the treatment site.
- The product comes into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination. These devices should be disposed of after use by following your facilities approved procedures for contaminated waste.
- CAUTION: US Federal law restricts this device to sale by or on the order of a physician
- Always hold the knife perpendicular to the cleaning block surface to prevent stressing the blade.
- DO NOT force or push diamond blade into the diamond knife cleaning block at an angle. Excessive forces may snap or detach the blade from the mount.

Incident Reporting:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established

HAZARDS ASSOCIATED WITH THE RE-USE OF SINGLE USE ONLY DEVICES:

- Single use devices have not been validated for re-use. If you re-use a device you may be held Legally Liable for the safe performance.
- Cross-contamination and infection risks due to unwanted exposure of bacteria, prions, fungus or virus.
- Device failure through material fatigue or degradation caused by initial use and design.
- May snap or cause blade to come out from housing.

Sterilisation:

The device is a SINGLE USE ONLY device supplied sterile and ready for use. Sterilisation is by Irradiation.

To use:

1. Thoroughly rinse the diamond knife with deionised water.
2. Soften one half of the block by moistening with sterile water.
3. Hold the moistened cleaning block between finger and thumb, DO NOT place on a hard surface
4. IMPORTANT – Use only a vertical up and down motion. Pierce the wet end of the cleaning block with the knife blade several times
5. Rinse the blade again in sterile water
6. Repeat the procedure on the dry half of the block to polish the diamond
7. Send for decontamination and sterilisation.

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